

Algilock Cleaner® Plus

Alginate remover

- Cleaning in dipping bath without mechanical impact, no ultrasound, no scratching
- Decomposes the organic structure of the impression material, no aggressive or caustic effect, no corrosion
- Even after several cleanings impression trays do not tarnish
- Bio-degradable and pH-neutral, phosphate-free, pleasant smell
- For approx. 500 trays

Easily removes plaster and alginates from impression trays, instruments and spatulas. No laborious and time-consuming cleaning, simply add 50 g to 1 l of warm water and leave for approx. 15 – 30 minutes. May also be used in an ultrasonic unit. Not suitable for aluminum trays.

<u>Article description</u>	<u>Content</u>	<u>REF</u>
Algilock Cleaner Plus	1000 g	554 104

