

Hydent[®] Sticks


*Color Transfer and Tissue
Marking Applicators*



Disposable, Bendable, Patient Friendly

100 Dual Bend Micro Applicators


- **Remove denture/appliance**
- **Locate problem area (high spot) on soft tissue**
- **Moisten problem area (have patient moisten area with saliva or if necessary apply a light water spray)**
- **Lightly contact problem area with bulb end of Hydent Stick**

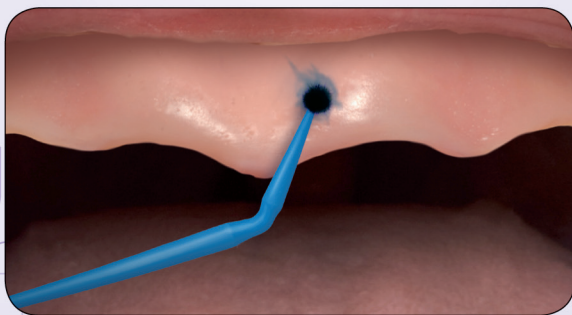


 **Pascal**[®]
International, Inc.

2929 NE Northup Way
Bellevue, WA 98004
425.827.4694
800.426.8051
pascaldental.com


- ▶ **Hydent Stick can be bent to reach difficult areas**

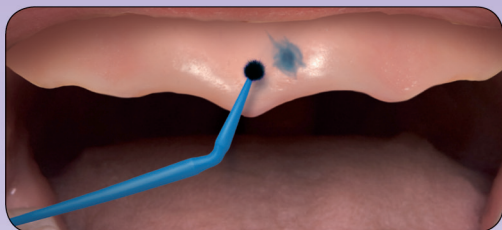

- ▶ **Insert denture/appliance then remove denture/appliance**
- ▶ **Locate color transfer high spot**

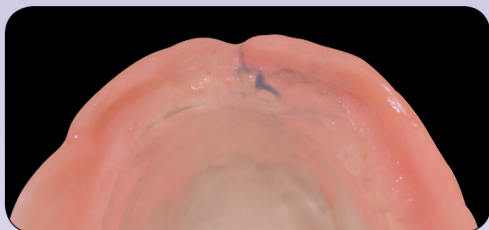

- ▶ **Remove color transfer high spot**

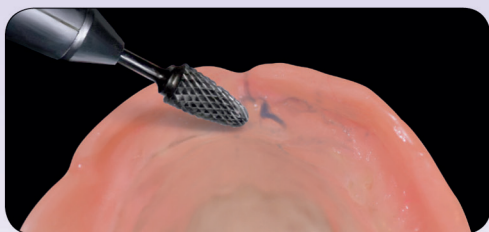

- ▶ **Re-Insert denture/appliance. Check for fit**
Repeat procedure if necessary

Place your order with your dealer or call
1-800-426-8051

